



Product Specification

cdhfinechemical.com

ED005

AIR AND SURFACE SPRAY DISINFECTANT SOLUTION FOR PREVENTING CROSS INFECTION

PRODUCT CODE	DI280
SYNONYMS	--
C.I. NO.	--
CASR NO.	--
ATOMIC OR MOLECULAR FORMULA	--
ATOMIC OR MOLECULAR WEIGHT	--
PROPERTIES	-
PARAMETER	LIMIT
Description	A clear & light blue or light green coloured liquid.
Solubility	Miscible with water.
Composition:	
Benzalkonium chloride solution	0.5% w/v
2-Propanol, perfume, permitted colours	70% v/v
Action	An air and surface sprayable disinfectant for preventing cross infections.
Use	Medical and Industrial
Medical	For disinfecting- Equipment's ECG monitors Ventilators, Respiration and AVD units. Fibrillators clinical thermometers, Stethoscopes, pulse oxymeters, incubators hand pieces, dental chairs, hospital beds, tabletops, Consulting rooms, Airspace between patients.
Industrial	In pharmaceutical , food and dairy industry for disinfecting laminar hoods, tabletops, workstations, Air and surface disinfection in critical areas..
Usage Directions	Spray undiluted on objects and surfaces that require disinfection until they are fully wet. Exposure time 1 minute for invisibly soiled areas and 15 minutes for visibly soiled areas. Spray undiluted aerially to contaminated rooms, clinicians chamber pathology laboratories research & development laboratories etc-until saturation.
Flammable	Keep away from heat & flame, avoid eye contact, In case of eye contact, rinse eyes thoroughly with water, In case of irritation, discontinue use & consult a Doctor.
Storage	Store in a cool, dry & dark place, protect from light.
Note(s): Assay (if applicable) method mentioned.	
WARNING -- Hazard statements: -- Precautionary statements-- Prevention: -- Response: --	IMDG Code :- UN No. :- IATA :-
Disposal: --	
Hazard Pictogram(s) :	